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OPERATING INSTRUCTIONS

**REBOOTS** **GOX** LITE

DE/EN/FR/ES/IT

# **INHALT**

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**ENGLISH**

# CONTENT

<b>Preliminary note</b>	<b>22</b>
<b>Package contents &amp; technical data</b>	<b>23</b>
<b>Explanation of the symbols</b>	<b>24</b>
<b>Extensions</b>	<b>25</b>
<b>Control unit overview</b>	<b>26</b>
<b>Overview LED status</b>	<b>27</b>
<b>Overview of pump unit</b>	<b>28</b>
<b>The first application</b>	<b>29</b>
<b>Overview of the Go X Lite programs</b>	<b>31</b>
<b>The Reboots app</b>	<b>32</b>
<b>Exiting the application</b>	<b>35</b>
<b>Troubleshooting</b>	<b>36</b>
<b>Safety instructions &amp; warranty</b>	<b>37</b>
<b>Liability notice</b>	<b>38</b>

## PRELIMINARY NOTE

Congratulations on purchasing this device! You will have a lot of fun with it! If you are satisfied with your Reboots product, we would be delighted to receive your recommendation.

These instructions contain details that should be observed by the user in order to prevent hazards and use the device safely.

If you have any questions about the application, accessories or suggestions, please do not hesitate to contact us!

**Telephone:** +49 (0) 666 174 891 06

**Whatsapp:** +49 1573 5990081

**Mail:** [hallo@reboots.de](mailto:hallo@reboots.de)

**Web:** [www.reboots.com](http://www.reboots.com)

**HelpDesk:** [help.reboots.com](http://help.reboots.com)

## BEFORE YOU GET STARTED

- Please read the operating instructions carefully before using the device.
- Keep these operating instructions in a safe place so that you can read them if necessary.
- The product must not be used under certain health conditions. Please read the list of contraindications on the last page carefully.

## PACKAGE CONTENTS & TECHNICAL DATA

Control Unit

USB-C Charging Cable

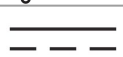
Go X Lite Pants (S/M or L)

Pump Unit (16 devices,  
pre-installed)

Number of programs	8
Number of air chambers	2x6
Weight [in kg]	4,2
Power output [in W]	45
Frequency [in Hz]	50 / 60
Voltage [in V]	220-240 AC
Time settings [in min]	5-99
Pump capacity [in l/min]	2
Pressure output [in mmHg]	30-180
Dimensions [in cm]	10 x 7.7 x 7.8
Protection classes	IP21; II/ <input type="checkbox"/>
Temperature range [in °C] (Operation and storage)	10 - 40
Humidity [in %] (Operation and storage)	30 - 85
Material	Nylon and polyamide fibers

## Explanation of the symbols

	Manufacturer
	Country of manufacture (China)
	Conformity symbol
	Class II equipment
	Refer to instruction manual
	Do not wash
	Use only inside
	Protect from rain
	Direct current
	Serial number
	Disposal instructions
	Disposal instructions Spain
	Disposal symbol for electrical appliances

	The product contains small amounts of lead. Follow the disposal instructions on page 38
	Disposal instructions Italy
	Disposal instructions France
	Instructions for disposal of packaging France
	Note on recycling of packaging


## EXTENSIONS FOR YOUR REBOOTS Go X Lite

### INCREASING THE WIDTH

If the width of your Reboots Go X Lite Pants is too small, you will notice that you cannot close the zippers or can only close them with great effort. In this case, you can use our extensions to create further width gradations. Just take a look in our [Online Store](#). The extensions have two further gradations so that we can solve the problem of increasing the width according to your individual needs.

# CONTROL UNIT OVERVIEW

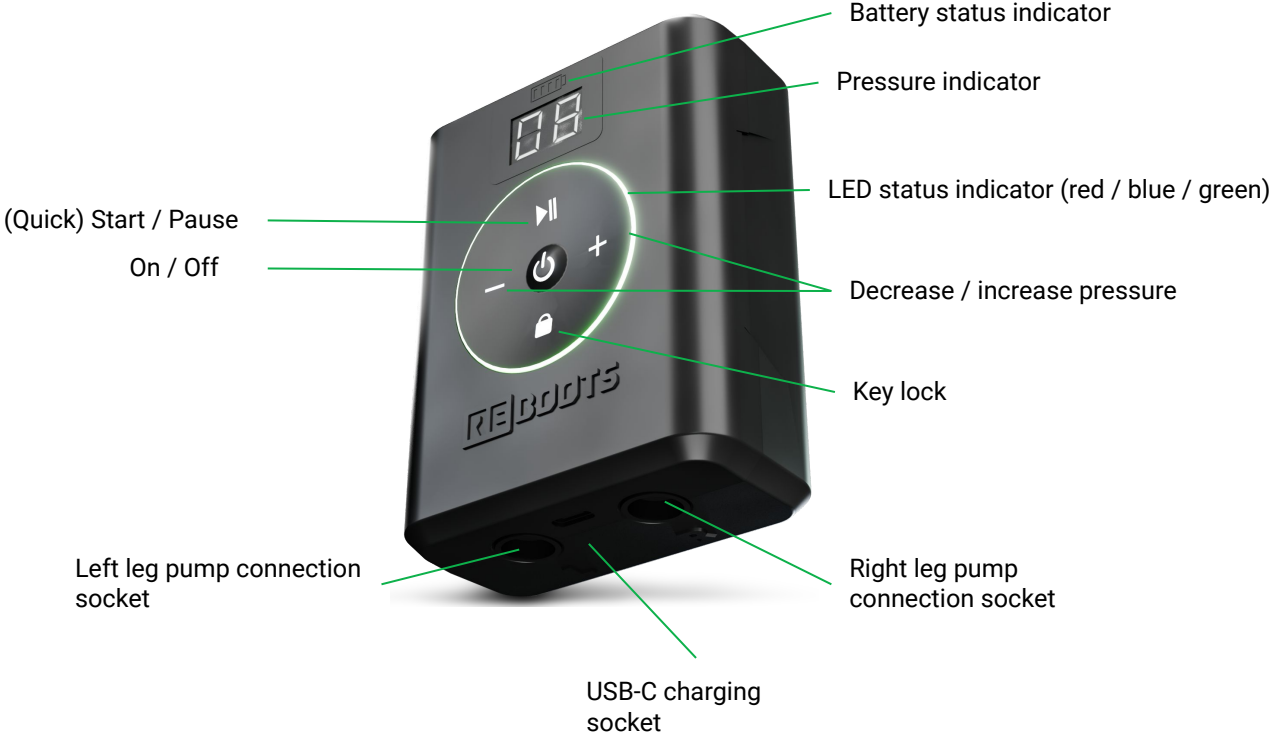## LED STATUS INDICATOR OVERVIEW

The LED status indicator lights up solid green:

- Go X Lite is ready for use
- Go X Lite is connected to your smartphone

The LED status indicator flashes green:

- Go X Lite is running the program you have selected

The LED status indicator lights up solid blue:

- Go X Lite is ready for use
- Go X Lite is searching for a Bluetooth connection

Note:


- You can also use Go X Lite with limited functionality without a smartphone

The LED status indicator lights up solid red:

- Go X Lite is not ready for use
- See the “Troubleshooting” chapter and contact our Support if necessary

# PUMP UNIT OVERVIEW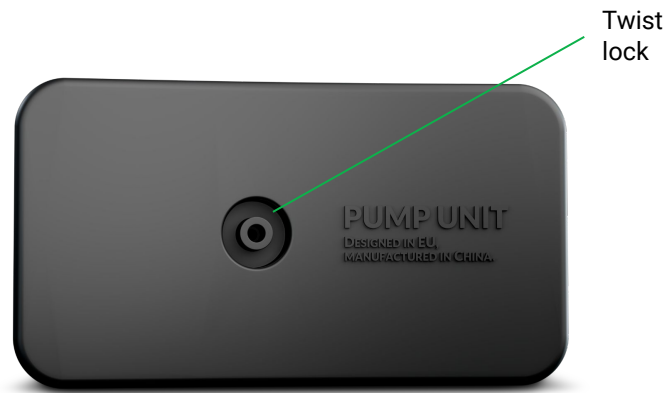

## PUMP UNIT FRONT

- On the front you will see a setup button
- This button is only needed when servicing (see Troubleshooting chapter #3 and #4)

## PUMP UNIT BACK

- On the back you will see the twist lock
- If you need to change a pump, you can simply detach it from the pants by turning it 90° counterclockwise
- Disconnect the power connection to the pump by pressing the pin on the plug and gently pulling the plug out

## USING THE PANTS FOR THE FIRST TIME - HOW TO PUT THE PANTS ON

### GETTING STARTED:

The easiest way to get started is to lay your Reboots Go X Lite Pants down on the floor. Open the zippers approximately to the beginning of the foot section for easy entry. Then lie down inside the Reboots Go X Lite Pants and close the zippers completely at the hips to prevent them from opening suddenly. Secure the waistband with the Velcro fasteners.

You can find more information regarding troubleshooting on page 36.

## USING THE PANTS FOR THE FIRST TIME

1

### CONNECT THE Go X Lite CONTROL UNIT TO THE PANTS & PUMPS

Attach your Go X Lite Control Unit to the pants by inserting the unit into the fastener on the pants and turning it 90°. If you have done everything correctly, the connections on the back of the Go X Lite Control Unit will now point towards the waistband.

Now insert the plug from the right leg into the "R" socket and the plug from the left leg into the "L" socket.

**Be very careful to connect it correctly, otherwise your Go X Lite Control Unit may become unusable.**

2

### SWITCH ON THE Go X Lite CONTROL UNIT

Switch on the Go X Lite Control Unit by pressing the "On-Off" button until you hear a beep.

3

### PRESS START

You're all ready to go. Press "Start" and enjoy your first massage with the Reboots Go X Lite.

You can adjust all the individual settings via your smartphone on our Reboots app. See the "Reboots app" chapter for details.

*Tip: To protect against soiling, we recommend wearing thin jogging pants or leggings. Make sure your pants pockets are empty. Also completely close the zippers of the Reboots Go X Lite Pants to prevent them from opening suddenly.*

# Go X Lite PROGRAM OVERVIEW

## MESSAGE PROGRAM A-H:

The air chambers in the Reboots Go X Lite Pants are filled according to the selected program. This creates an optimal gliding pressure wave massage for a relaxing feeling on both legs. The duration and pressure, and all optional additional functions, can also be changed during the massage

A	Intense Flow
B	Intense
C	Impulse
D	Flow
E	Switch
F	Double
G	Intense - Pulse
H	Intense - Double

**A** Intense Flow

**B** Intense


**C** Impulse

**D** Flow

**E** Switch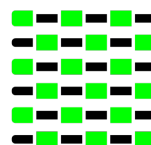

**F** Double


**G** Intense Pulse ★

**H** Intense Double

# YOUR OPTIONS WITH THE REBOOTS APP
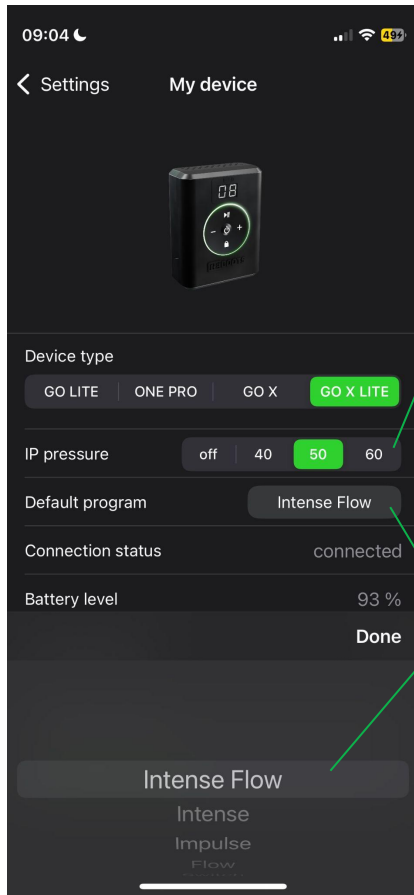
## IP PRESSURE FEATURE


- Before your selected program/sequence starts, we want to make sure that the Reboots Go X Lite Pants have adapted to your legs in the best possible ergonomic way. To do this, all chambers fill with air simultaneously up to the desired pressure level of 40, 50 or 60 mmHg. Your program/sequence will then start. This feature is only active once immediately after starting the control unit and is only active again after restarting the device.
- If you don't want to use the Initial Phase, you can of course deactivate it.

## ASSIGNING A DEFAULT PROGRAM TO THE QUICK START BUTTON

- If you need to get started quickly and want to use your Reboots Go X Lite with your favorite Reboots program without using your smartphone, you can select your favorite program from a list. After you have restarted the Reboots Go X Lite Control Unit, your favorite program is saved and will be activated as soon as you press Start - even without using your smartphone.

# YOUR OPTIONS WITH THE REBOOTS APP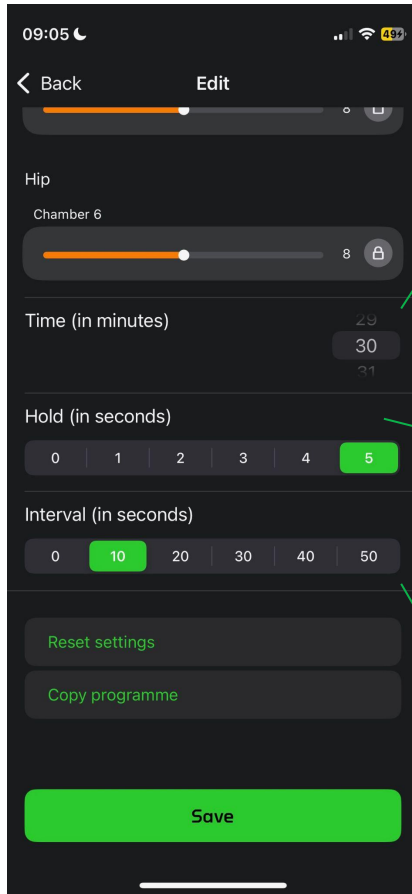

## SET THE TIMER

- Select the running time for the program (5-99 min.)

## SET THE HOLD TIMER (HOLD)


- This holds the target pressure in the active chamber(s) for a defined period
- The hold timer will start when the target pressure in the chamber(s) is reached
- By pressing the button, the time period can be set from 0-5 seconds in 1-second increments

## SET THE INTERVAL TIMER (INTERVAL)

- Set the pause time between two massage intervals
- The interval timer will start after the hold time for the last chamber(s) of the message interval of any program has expired
- By pressing the button, the interval can be set from 0-50 seconds in 10-second increments


# YOUR OPTIONS WITH THE REBOOTS APP

## SET THE INDIVIDUAL TARGET PRESSURE FOR EACH CHAMBER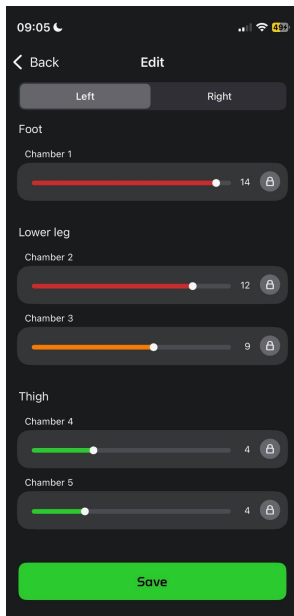

### IN THE EDIT MENU

In each program, you can set the pressure for each chamber by using the horizontal slider. If the lock symbol is closed, the settings apply to the corresponding chambers on both legs. If you open the lock, you can set the chambers separately for each leg. Your settings are saved permanently.

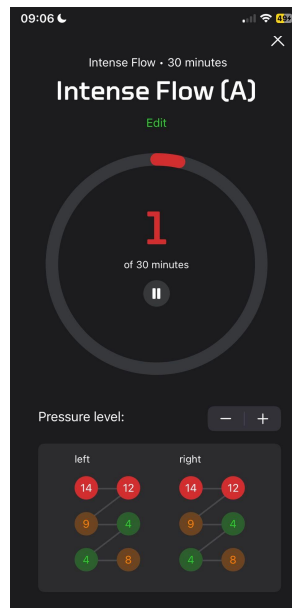### WHEN OPERATING THE PROGRAM

If you have already started the program, you can easily change the pressure in each chamber by tapping the corresponding chamber and changing the pressure by using the vertical slider. The settings will be lost after the program has finished. To save the settings permanently, simply use the Edit menu.

## ENDING THE APPLICATION

### THE PROGRAM / SEQUENCE HAS ENDED AUTOMATICALLY:

When your session has ended, the pumps switch off and release the air so that you can exit the Reboots Go X Lite Pants .

### THE PROGRAM / SEQUENCE HAS ENDED MANUALLY:

If you have ended the current session via the app or the Start-Pause button or on the Reboots Go X Lite Control Unit by pressing and holding the On-Off button, the pumps will switch off and release the air so that you can exit the Reboots Go X Lite Pants.

### THE PROGRAM / SEQUENCE WAS PAUSED MANUALLY:

If you have paused the current session via the app or the Start-Pause button, the air will remain in the chambers until you end the program in the app or switch off the Reboots Go X Lite Control Unit by pressing and holding the On-Off button.

### EXIT:

Open the Velcro fasteners on the waistband and then open the zippers down to near your ankle to exit the Reboots Go X Lite Pants comfortably and easily.

Then open the Reboots Go X Lite Pants. To deflate the air chambers quickly after use, press on the air chambers with both hands.

### STORAGE:

To store your Reboots Go X Lite Pants, fold them loosely together. Follow the storage instructions on the next page.

### CLEANING:

All you need to clean the Reboots Go X Lite Pants is a damp cloth with a gentle cleaning agent. Allow the Reboots Go X Lite Pants to dry thoroughly.

### DISINFECTING:

Disinfectants such as Descosept pure or other products with a similar composition can be used to disinfect your Reboots Go X Lite Pants.

# TROUBLESHOOTING

## #1 THE REBOOTS Go X Lite CONTROL UNIT CANNOT BE SWITCHED ON

Check to see if the battery is sufficiently charged. Use the USB-C charging cable (included) with a suitable USB-C charger to charge the battery. You can, of course, use your Reboots Go X Lite while charging.

## #2 INDIVIDUAL CHAMBERS ARE NOT INFLATED EVEN THOUGH THE LED STATUS INDICATOR IS GREEN

Check to see if you have disabled individual chambers in the app. If you have not disabled them, continue as follows.

Make sure that all pumps are properly connected to the chambers. To do this, open the black service zippers located next to the large zipper on the pants and check that the pump units are attached properly. The pump units can be removed and attached in the same way as the Go X Lite Control Unit, by using a twist lock.

## #3 THE ORDER OF THE CHAMBERS DOES NOT MATCH THE DISPLAY ON THE APP

The order of the pump units may have been programmed incorrectly. Contact our Support Team. We will solve the problem with you by phone and/or by video.

## #4 THE LED STATUS INDICATOR FOR THE REBOOTS Go X Lite IS LIT UP SOLID RED

The Reboots Go X Lite Control Unit is not detecting individual pump units or all pump units. You may have accidentally plugged the cable connections on the legs into the connection sockets the wrong way around. This can lead to a serious error in the Reboots Go X Lite Control Unit. In this case, please contact our Support.

## #5 THE REBOOTS Go X Lite CONTROL UNIT BEEPS BRIEFLY AND THEN CANCELS THE MESSAGE

The Reboots Go X Lite Control Unit has lost the Bluetooth connection to the smartphone. When this happens, the message is interrupted for safety reasons. Reconnect and keep the smartphone near you within a range of approx. 5 meters.

## #6 THE REBOOTS PANTS ZIPPER OPENS DURING USE

To fix this, pull the zipper all the way up and fold the slider handle towards your feet. This engages a mechanical lock that prevents the pants from opening. After use, you can operate the zipper as you normally would to get out.

# SAFETY INSTRUCTIONS

Keep these operating instructions in a safe place! Do not disassemble or incinerate the device. Repairs to the control unit may only be carried out by specialists authorized by Wellcosan GmbH, otherwise the warranty may be voided. If the appliance is damaged, unplug it from the mains and switch it off. Also unplug the appliance and switch it off if it has come into contact with water or other liquids. Do not use the device in damp rooms. Ensure that the appliance is stable. Do not subject the device to shocks and do not throw it. The device must not be covered by other objects during operation. Ensure that the hoses do not twist or kink during use and storage. Keep the appliance away from sources of heat (e.g. direct sunlight, radiators and others). The appliance must be switched off when not in use. The device switches itself off after 10 minutes of inactivity.

## WARRANTY

We grant a 24-month warranty on the control unit. In the event of a fault, it may be necessary to return the device for inspection. Please ensure that the shipping carton is suitably padded to prevent any damage during transportation. Unfortunately, no warranty claims can be made for defects caused by improper shipping.

**Warranty conditions:** Insofar as there is a legal obligation to provide a warranty, either a replacement shall be provided free of charge or the manufacturer shall be given the opportunity to rectify the defect, excluding the right to rescind the contract or reduce the purchase price. If the repair is unsatisfactory despite several attempts, or if a replacement device also has a defect for which the manufacturer is responsible, the customer has the right to cancel the contract or reduce the purchase price.

Also unplug the appliance when not in use. Always remove the plug from the socket before cleaning the appliance. Keep the product away from children or persons who have not been instructed in its use.

With the CE marking, the manufacturer or importer confirms the conformity of the product with the applicable EC directives and compliance with the "essential requirements" specified therein.

This marking on the device means that you must not dispose of it in normal household waste at the end of its service life. Please have it disposed of by an electronic waste disposal company.

The abbreviation UKCA stands for UK Conformity Assessed. The marking is a prerequisite for the marketing of products within Great Britain.

**Warranty conditions:** Excluded from the guarantee are defects caused by violent damage, improper operation, external force or by modifications and repairs carried out by third parties, e.g. defects caused by incorrectly dimensioned or short-circuited fuses or defects caused by normal wear and tear.

**Warranty conditions:** The warranty claim is only valid in conjunction with the original proof of purchase (invoice). Please keep the proof of purchase in a safe place.

If you have any problems or questions when using the system, please do not hesitate to contact us on +49 (0) 666 174 891 06 or hallo@reboots.de.

# LIABILITY NOTICE

Ideally, only use your Reboots product after consulting your doctor.

**Possible contraindications** may include Decompensated heart failure, cardiac insufficiency with water in the legs, inflammation of the treated body parts, known or suspected leg vein thrombosis, phlebitis and arterial diseases, gangrene, dermatitis, arteriosclerosis or venous circulatory disorders, open, untreated or infected wounds and injuries, pain in the legs, skin injuries, acute inflammation, malignant tumors, pulmonary oedema, high blood pressure, extensive thrombophlebitis, thrombosis or suspected thrombosis, erysipelas, severe uncontrolled hypertension, acute soft tissue trauma to the extremities, neuropathy or occlusive processes in the lymphatic drainage area.

**Intended use:** Your Reboots product is intended to help you feel better and relax.

## **DISPOSAL:**

Please note that the product contains small amounts of lead. Therefore, please have it disposed of by an electronic waste disposal company. **IMPORTANT:** Please do not dispose of the product with household waste!

**ALTERNATIVE:** We will be happy to dispose of your old reboots free of charge. Just send them to us:

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Please enclose a note requesting disposal.

## **NOTE:**

Your Reboots product is not a medical device and is not intended for self-treatment. If you are unsure, please ask your doctor.

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© Wellcosan GmbH 2024
Wellcosan GmbH, Hauptstraße 27